



Easy-Action™ Ring Applicator (Part # 1265FG)




Instructions for Use

Designed for use with the
Osbon ErecAid® Esteem®
Vacuum Therapy Systems

- Automatic System
- Manual System
- Classic OTC System




How to Use the Easy-Action™ Ring Applicator

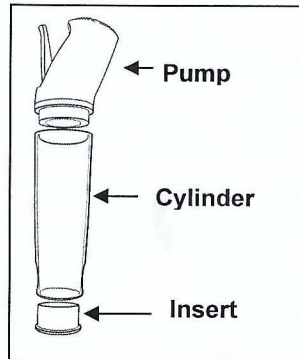


Step 1

Remove the pump and the insert from the cylinder. Put a towel on a table or counter top and stand the cylinder on top of the towel with the large end of the cylinder face down. The towel will keep the cylinder from slipping.

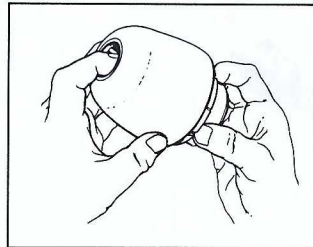


Step 2

To separate the Easy-Action™ Ring Applicator, hold the base of the cone in your hand and then **PUSH THE BUTTON** on the top of the cap. Place the base of the Ring Applicator cone in the small open end of the cylinder. Twist the cone so that the white dot on the top of the cone lines up with the cylinder groove opposite the cylinder label.

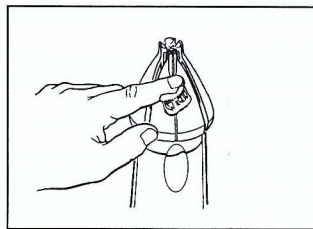



Step 3

GENEROUSLY lubricate the entire surface of the cone with **Osbon Personal Lubricant™**. Lack of lubricant will result in damage to the tension ring.

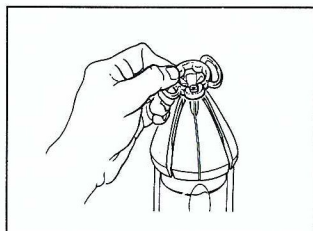


Step 4

Place the tension ring on the cone, making sure to align the **U-notch** on the tension ring with the **white dot** on the cone.

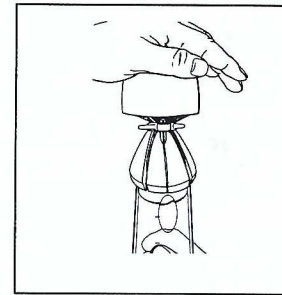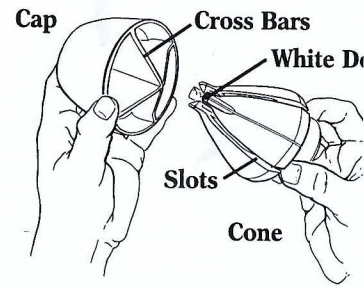


Step 5

Place the cap back on the cone with the tension ring, lining up the cross bars on the underside of the cap with the slots on the top of the cone.

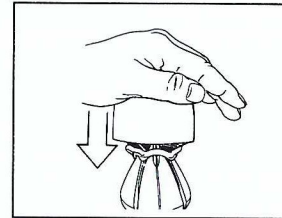


Step 6

QUICKLY and **FIRMLY** press straight down on the cap of the ring applicator with one hand while holding the cylinder in an upright position on the towel with your other hand to help stabilize it.

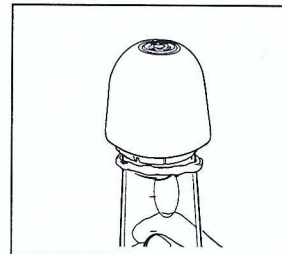

Step 7

The tension ring should now be transferred to the cylinder in the correct position. If you need to add additional rings to the cylinder, slide the first ring 1/4" (6mm) towards the pump end of the cylinder to allow space for the additional ring. Then, choose another ring and repeat steps 2 through 6.

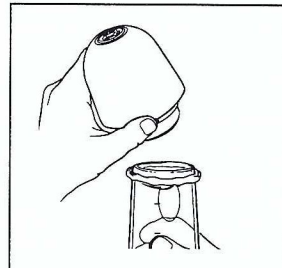

Step 8

After you have loaded your tension ring(s), remove the Ring Applicator from the cylinder. You are now ready to create an erection with your ErecAid® System.

Step 9

After use, wash cone and cap well with warm soapy water to remove lubricant. Set on towel to dry. Store in carrying case.