

OSBON ERECAID RING TENSION CHART

	Level One	Ring A	Large Beige Standard Tension Ring
	Level Two	Ring B	Large Pink High Tension Ring
	Level Three	Ring C	Medium Beige, Standard Tension Ring
	Level Four	Ring D	Medium Pink, High Tension Ring
	Level Five	Rings A & C	Large Beige Standard Tension Ring Medium Beige, Standard Tension Ring
	Level Six	Rings B & D	Large Pink High Tension Ring Medium Pink, High Tension Ring
	Level Seven	Rings C & D	Medium Beige, Standard Tension Ring Medium Pink, High Tension Ring



A Ring



B Ring



C Ring



D Ring

You will receive all 4 rings when you purchase your Osbon ErecAid System (A, B, C & D).

The inside diameter of the large rings is 22mm (0.875")

The inside diameter of the small rings is 19mm (0.750")

When using two rings, the ring with the tightest tension should be closest to the body.

This means that when you are loading rings onto the cylinder, you will need to load the loosest ring first, followed by the tightest ring.

Additional rings are available for purchase (not included with ErecAid Systems) are Medium and Large Maximum Elasticity Rings. These rings are looser than the Beige A & C rings for men who require slightly less tension.

