

8. To remove spent syringe, unscrew injector and carefully pull syringe out of lower portion.
9. Screw injector back together and place in storage (see figure a).

*Always dispose of used syringes as directed by your medical professional.*


Patent #6,203,530

### EZ Auto Inject™ Sleeve

The depth of the needle penetration is determined by the sleeve length. The Green sleeve, included with your EZ Auto Inject™, will provide an approximate needle depth of 9.5mm[3/8”.

Sleeves are inserted into the small end of the lower portion of the injector, green stripe facing in, push until it snaps securely in place (see figure h). The sleeve is now permanently installed.


Fig. h

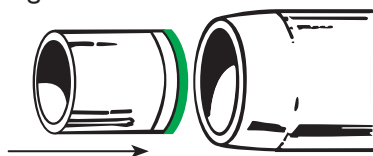

Sleeve	Syringe Type and Style	Approximate Needle Depth
Green	B-D - 1cc	9.5mm/3/8”

Caution: Federal law restricts this device to sale by or on the order of a physician licensed by the law of the state in which he or she practices to use or order the use of the device and the method for its application or use.

# EZ Auto Inject™

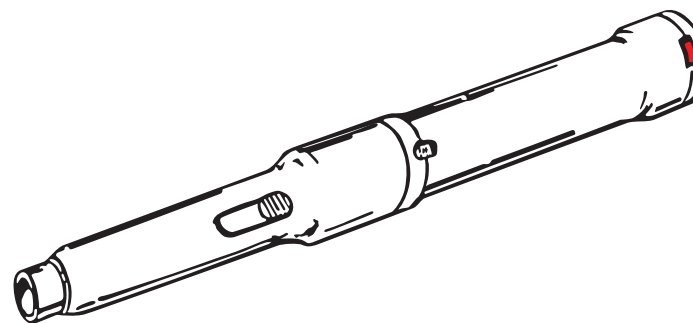

## USER GUIDE



TIMM Medical Technologies, Inc.

2111 W Wyatt Earp | Dodge City, KS 67801 | 800.438.8592 | www.timmmedical.com



## Instructions for use of EZ Auto Inject™


User must read **ALL** instructions very carefully before attempting the procedure. Always follow instructions given by medical professionals.

For Single Patient Use Only.


Use only B-D and Teruma 1ml or lcc syringes either Insulin (fixed needle) or Tuberculin (replaceable needle tip) style.

Questions? Call 1.800.438.8592


1. Unscrew upper and lower portions of injector and separate (see figure a).

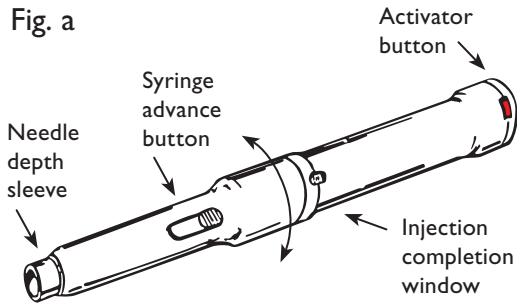

2. Prior to setting the injector for activation, insert the needle depth sleeve (see fig. h). To activate injector, turn lower portion around and insert small end into upper portion, pushing the spring until you hear a click and feel the latch engage. Then remove the lower portion from upper portion (see figure b).

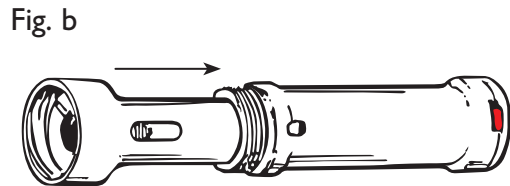


3. Clean needle depth sleeve by wiping inside and out with alcohol wipe. Now insert previously filled syringe in lower portion by placing sterile cover of syringe into the center of the large end of the lower portion of injector (see figure c).

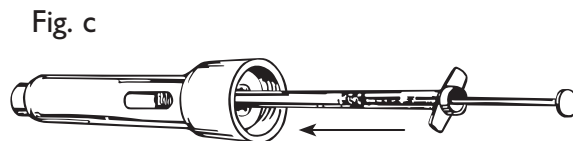


Push syringe into lower portion of the injector until it is completely installed in the injector and syringe tabs bottom out against internal components of injector, trying not to apply pressure to syringe plunger.


4. Align the black arrow with the injection completion window. Screw the two portions of the injector together. The injector is now activated and loaded with safety on (see figure d).
5. Prepare the injection site as per the instructions given by your medical professional.

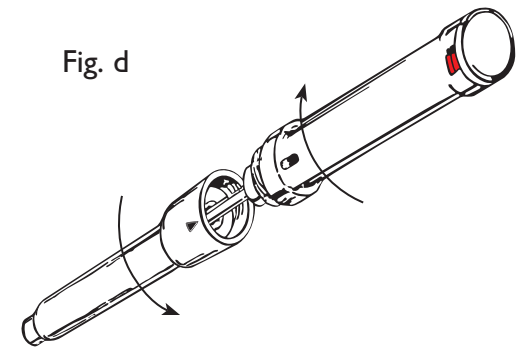

6. Remove the sterile needle cover by pushing the syringe advance button forward until the needle cover is exposed. You must push and hold the advance button in place. Carefully grasp the needle cover with your free hand and remove, exposing the needle (see figure e). Allow the syringe advancing button to return to its original position. The syringe needle is now retracted and should not be visible.

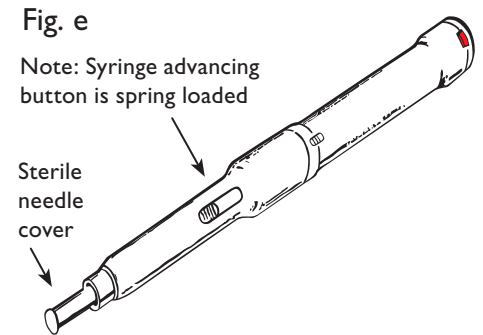

7. Apply the injector to the injection site, lightly pressing it against your skin. Slightly depress and slide activator button to the left as shown by the arrow, injection will occur instantaneously (see figure f). Medication is fully dispensed when colored indicator stripe shows in completion window. Hold injector steady until syringe is emptied (approximately 10–15 seconds).

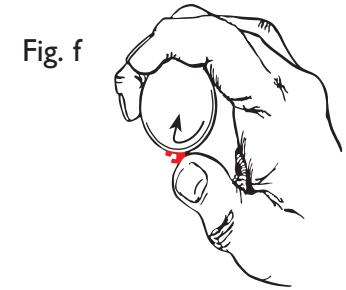

When injection is complete lift injector away at 90° right angle to injection surface (see figure g). Attend to injection sight as directed by your medical professional.

