

# USER'S MANUAL

## Ring Kit

### POS-T-VAC MEDICAL

2111 W Wyatt Earp • PO Box 1436

Dodge City, KS 67801

Toll Free: 1-800-279-7434

Fax: 620-227-8474

Rejoyn Ring Kit Contents.....	1
Warnings & Precautions.....	1, 2
Risks.....	3
How To Use The Ring Kit.....	5
Concerns.....	6

## Table of Contents

**NOTE: Carefully read instructions completely before use of the device.**

## RING KIT

The support ring kit is used to maintain penile rigidity in men with erectile dysfunction by restricting penile venous outflow after the patient has obtained an erection (either with the aid of a vacuum pump or naturally). The support ring is placed around the base of the erect penis to restrict the flow of venous blood leaving the penis.

The Ring Kit consists of:



Other accepted forms of treatment for Impotence are:

1. Counseling
2. Hormonal Therapy
3. Medications (oral and injectable)
4. Intraurethral
5. Vascular Reconstructive Surgery
6. Prosthetic Implants (surgical)

Any of the above listed treatments should be under a doctors care and/or by prescription.

We are always happy to discuss any problems that you may have with the use of the Rejoyn System. If you have questions or would like to order replacement parts, please call our user help line 1-800-279-7434.

**WARNING:** Consult your doctor should any problems occur, and stop using the device if such conditions continue.

**WARNING:** If you have symptoms of erectile dysfunction, (i.e., unable to get an erection that is good enough for sexual intercourse), talk to your doctor prior to using this device to avoid a possible harmful delay in figuring out any of the most common causes of this condition, such as diabetes, multiple sclerosis, cirrhosis of the liver, chronic renal failure, or alcoholism.

**WARNING:** If you do not heed the warnings below you may cause personal injury

1. Remove support ring within thirty (30) minutes. If worn longer than 30 minutes at a time, the support ring may cause permanent injury to the penis.

2. Frequent use of support rings may result in bruising at the base of the penis (where the shaft of the penis meets the pubic area).

## HOW TO USE YOUR RING KIT



1. Place a generous amount of water soluble lubricant on the entire surface of the cone. Grip each handle, and press the support ring down over the cone onto the loading sleeve, using your thumb nails as shown in drawing.

1. Allow a minimum of 60 minutes between uses. If worn longer than 30 minutes at a time, the support rings may cause permanent injury to the penis.
2. Do not fall asleep wearing the support ring.
3. Support rings should not be used as a birth control method.
4. Use the largest ring which holds an erection.
5. Do not use support rings while using alcohol or drugs.
6. Frequent use of support rings may result in bruising at the base of the penis (where the shaft of the penis meets the pubic area).
7. Misuse of the support rings may worsen already existing medical conditions such as Peyronie's disease ( a condition that causes distortion or abnormal bend of the erect penis), Priapism (prolonged erection), and blocked urethra (urinary tract).
8. Do not use if you have decreased sensation of pain in the area of the penis.
9. Do not use if you have decreased hand strength that will make the removal of the rings difficult.
10. Do not use oil based lubricants they will be harmful to the rings and other latex type material such as condoms.
11. Use of the support ring may cause bruising. Do not be overly concerned if you notice some bruising of the penis or small red spots below the skin. Use caution and plenty of water soluble jelly when removing the support ring.

## Don't Be Overly Concerned If...

Your penis turns slightly bluish in color after you apply the support ring. This may be a normal response to the increase of blood in the penis. The bluish color should disappear when you remove the support ring.

You or your partner notice a slight drop in the temperature of the penis. This coolness is normal and not harmful. It is a normal response to the reduced blood flow caused by the support ring.

Your erection has a "hinge" effect. The penis may pivot at its base when the ring is in place. If this occurs, you may find it necessary to try different positions with your partner for successful intercourse.

Your semen does not flow through at point of climax. The pressure of the support ring may interfere with the ejaculation process. This is not harmful, and does not affect climax. Your semen may flow out when you remove the ring. Support rings are not a means of contraception or birth control and should not be used to avoid pregnancy.

2. Separate the cone from the loading sleeve.
3. Slip the loading sleeve over the penis and slide the ring off the loading sleeve onto the base of your penis as close to your body as possible.
4. The ring should grip the base of your penis comfortably.
5. If you get a good firm erection and then it slowly goes down, you may need to use a smaller ring.
6. Remove the ring by pulling outward on the side handles. Following use, wash, dry and return the ring to its plastic bag. Make sure the bag is air free and tightly sealed for longer life.
7. If you are unable to get a good, firm erection after following steps 1 through 5, and your erection is not firm enough for satisfactory vaginal entry when properly lubricated, you may need to order the Rejoyn Vacuum System. Call our Toll Free number 1-800-279-7434 for more information.